


VEYPRSUB4

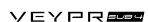





MADE TO



SUB







A 10 YEAR DREAM

MADE TO MEASURE THE PERFECT FIT

Only RGK offers the 'real' made to measure to create the perfect wheelchair for you. Choose optimum posture, comfort and performance. The 'real' made to measure is unlike any other and allows you to adjust the characteristics of your wheelchair to suit your individual needs, ensuring you can enjoy an active lifestyle.

- OVER 30 INDIVIDUAL MEASUREMENTS
- A RANGE OF ERGONOMIC OPTIONS
- LIGHTWEIGHT AND COMPACT DESIGN FEATURES

SEAT TAPER



DAVID'S **PERFECT** FIT

> ERGONOMIC BACKREST

> > ERGONOMIC SEAT



ULTRA-LIGHTWEIGHT SUB4 GUARANTEED

A lighter wheelchair means everyday actions become easier. With the Veypr Sub4 transferring into the car and pushing long distances is effortless. This is the ultimate wheelchair for an active lifestyle.

- SUB4 TECHNOLOGY "
- ~10% LIGHTER THEN AN OCTANE SUB4
- FROM ~2.4 KG FRAME WEIGHT
- FROM ~3.3 KG TRANSPORT WEIGHT

- Excluding wheels, cushion and non-certified options.
- ** Refers to RGK's redesigned componentary that ensures all Sub4 wheelchairs are > 4 KG *





BUILT FOR EVERYDAY ADVENTURES PUSH THE LIMITS

With a high everyday usability the **Veypr Sub4** is built for everyday adventures. With its captivating appearance and ergonomic design the **Veypr Sub4** provides the ultimate rider experience.

"IT'S UNBELIEVABLE"

DAVID WEIR



KEY FEATURES

CARBON FIBRE

HERRINGBONE WEAVE PATTERN

FRAME

The final layer of carbon fibre is hand laid with precision, achieving the perfect herringbone pattern.



SUB4 UPHOLSTERY

ERGONOMIC SEATRAIL TUBE

Expertly engineered for comfort and ease of transfer.



CARBON FIBRE CASTOR FORK

~ 40% weight saving compared to its aluminium equivalent.



CARBON FIBRE FOOTREST

SUL

EXCEPTIONAL PERSONALISATION MAKE IT YOURS

Put your stamp on it, your colour, a personal detail, the **Veypr Sub4** decals are personalised to suit you.






FRAME FINISH SATIN OR GLOSS



CASE STUDY DAVID WEIR'S VEYPR SUB4

MEET DAVID WEIR CBE

David Weir is a 6 time Paralympic gold medallist and co-founder of the Weir-Archer Academy. He is a super star athlete, role model and is arguably one of the greatest wheelchair racers and Paralympian's of all time. We are honoured that he is sitting in a Veypr Sub4 now.

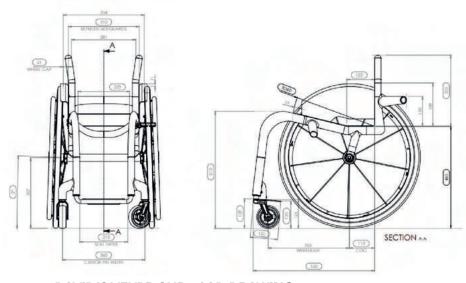

DAVID'S VEYPR SUB4 CAD DRAWING



WHAT WAS THE END RESULT?

"My Veypr Sub4 is so much lighter than my previous chair and the ride is unbelievably smooth. The ergonomic options have improved my seating position and posture and the extensive made to measure process has ensured the frame fits me perfectly.

It's made a huge difference in my day to day life, I didn't expect what I thought were such little details to make such a huge difference to my everyday."

- David Weir

TECHNICAL STUFF

| DESCRIPTION | MIN | MAX |
|--|--------------|---------------|
| Overall length with leg rest (no wheels) | 390 mm | 625 mm |
| Overall width (seat width) | 250 mm | 440 mm |
| Folded height (only for chairs with folding backs) | N/A | N/A |
| Total mass (with wheels) | 5 kg | 15 kg* |
| Mass of the heavist part (transport weight) | 3 kg | 12 kg* |
| Static stability downhill | N/A | 5° of incline |
| Static stability uphill | N/A | 5° of incline |
| Static stability sideways | N/A | 5° of incline |
| Seat plan angle | 0° | 45° of tilt |
| Effective seat depth | 250 mm | 440 mm |
| Seat surface height at front edge | 380 mm | 520 mm |
| Backrest angle | -10° of tilt | 20° of tilt |
| Backrest height | 150 mm | 400 mm |
| Footrest to seat distance | 200 mm | 520 mm |
| Leg to seat surface angle | 70° of angle | 90° of angle |
| Armrest to seat distance | N/A | N/A |
| Front location to armrest structure | N/A | N/A |
| Hand rim diameter | 24° -in | 25° -in |
| Horizontal location of axle | 25 mm | 125 mm |
| Minimum turning radius | 270 mm | N/A |

Note: The above is a guide. For any requirements outside of this table may result in a design change.

^{*}Based on additional options, excluding third party wheelchair add-ons.



www.rgkwheelchairs.com

RGK Wheelchairs Ltd.
Unit 8a/b/c Ring Road Zone 2,
Burntwood Business Park,
Burntwood, Staffordshire, WS73JQ, UK

Phone +441543670077 Email: info@rgkwheelchairs.com

